


Neotech Cell WipeTM and Cell ShieldTM

Cell Phone and Mobile Device Wipe and Cover




Neotech Cell Wipe™

Cell Phone and Mobile Device Wipe

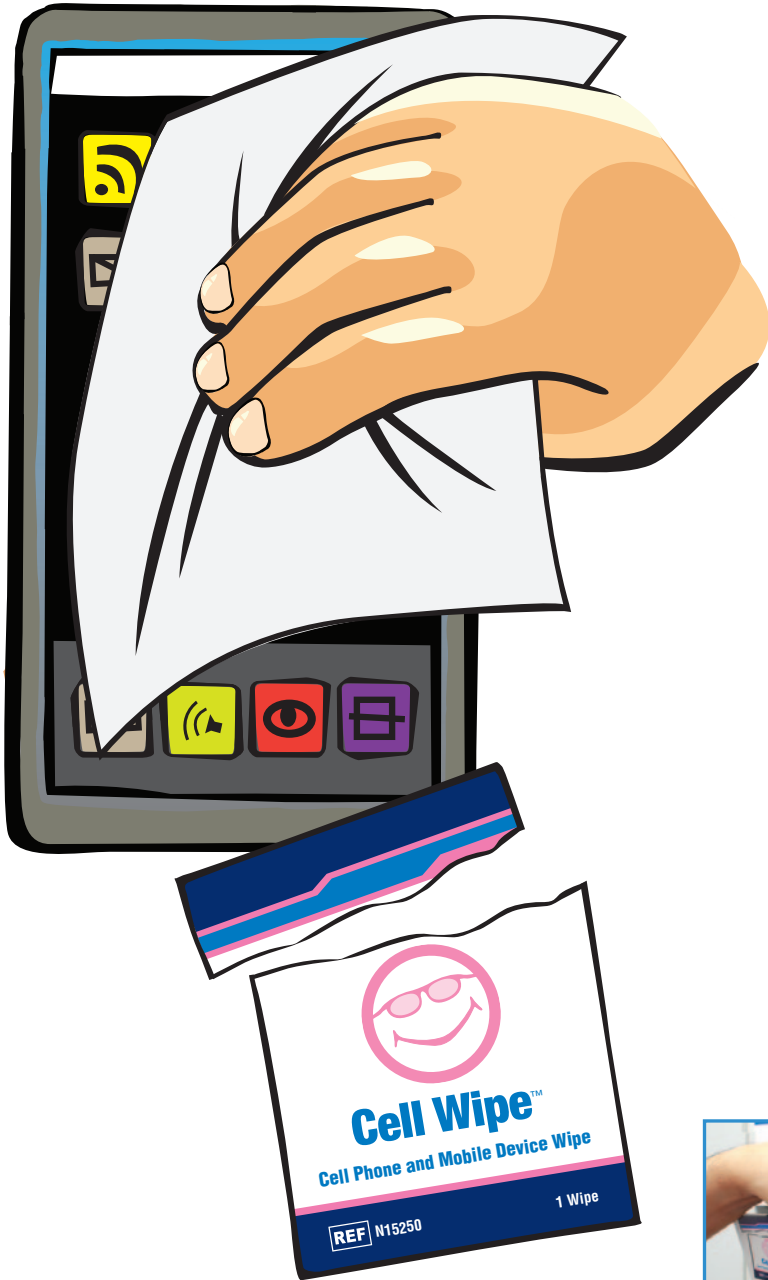

FEATURES AND BENEFITS

- Leaves no streaks or lint behind
- Pre-moistened and non-abrasive
- Eliminates smudges, fingerprints and oils
- Disposable
- Ready to use dispenser box
- Individually packaged

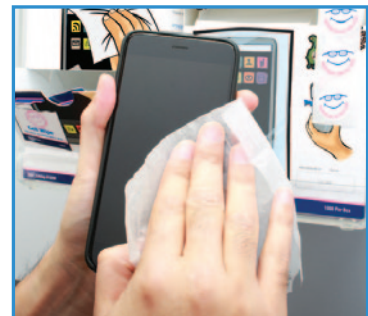

New size available

Neotech Cell Shield™

Cell Phone and Mobile Device Cover



FEATURES AND BENEFITS

- Helps to keep cell phones and other mobile devices clean
- Touch, talk, swipe and take pictures right through the clear cover
- Disposable
- Ready to use dispenser box
- Not made with natural rubber latex or plasticizer DEHP

Cat No.	Item	Qty/Unit
N15250	Cell Wipe	1000/case*
*4 boxes of 250		
New Large Cell Shield (120 mm x 225 mm)		
N15100	Combo Pack with Large Shield	400ea/case**
**4 boxes of 100 Cell Shields & 100 Cell Wipes		
Original Cell Shield (90 mm x 185)		
N15001	Original Cell Shield	1000/box
N15111	Starter Pack	1/box†
† Includes Housing, 1 box Original Cell Shield (1000), 1 box Cell Wipe (250)		
N15900	Acrylic Housing (no product)	1/box



Neotech Cell Wipe™ and Cell Shield™

Cell Phone and Mobile Device Wipe and Cover

INTENDED USE

The Neotech Cell Shield is intended for use as a temporary cover to help keep cell phones and other mobile devices clean. The Neotech Cell Wipe is intended for use to clean cell phones and other mobile devices.

DIRECTIONS FOR USE

Step 1 Tear open Cell Wipe packet, remove and unfold wipe.

Step 2 Gently wipe surface of cell phone or mobile device as shown. Dispose of wipe in trash.

Step 3 Wash or sanitize hands.

Step 4 Pull out and tear off one cover and one sticker from dispenser box.

Step 5 Insert cell phone or mobile device into cover, camera end first as shown.

Step 6 Fold over open end of cover toward back side of device.

Step 7 Remove sticker from backing and affix as shown, to hold cover closed.

Step 8 Replace Cell Shield every 12 hours or sooner if visibly soiled or torn. Discard after use.

GENERAL INFORMATION

Cell Wipe

Ingredients

Isopropyl alcohol, water, sodium laureth sulfate.

Product Material

32gsm 20# Wet Strength

Size

6 in. x 5 in.

Packaging

Individually packaged: Yes

Package type: Foil

Dispenser box: Yes

Cell Shield

Product Materials

Original Cell Shield (N15001, N15111):

Biodegradable polyethylene

Large Cell Shield (N15100): Recyclable polypropylene retail film

Sticker: Paper adhesive

Made with natural rubber latex: No

Made with plasticizer DEHP: No

Consult MSDS for complete material information

Size

Original Cell Shield (N15001, N15111):

90 mm x 185 mm

Large Cell Shield (N15100): 120 mm x 225 mm

Sticker: 25.5 mm

Packaging

Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information or to download the Material Safety Data Sheet (MSDS).

Boxes are printed with environmentally friendly soy ink on recyclable material.

CAUTIONS

Not intended for medical use.

Cover not intended to protect devices from water or other damage.

Do not use wipe to clean skin.

Do not expose wipe to open flame.

In case of eye contact, immediately flush eyes thoroughly with water.

Keep out of reach of children.

Always follow device manufacturer's guidelines.

Do not dispose of in toilet.

Single use only.



**For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com**

Neotech Products, Inc. • 28430 Witherspoon Parkway • Valencia, CA 91355