


NeoBar[®]
ET Tube Holder

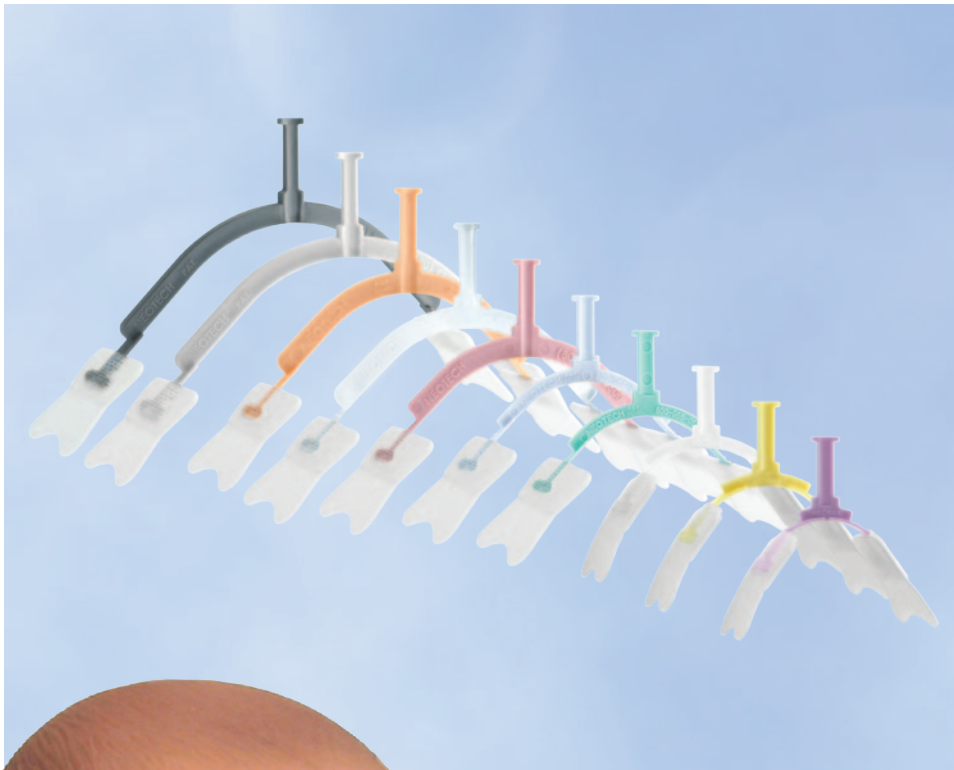

FEATURES AND BENEFITS

- Skin friendly NeoBond[®] Hydrocolloid tabs
- Macro and Jumbo sizes available with medical grade acrylic adhesive
- Designed to reduce extubations, help prevent palate trauma and allow for better oral care
- Eliminates need for tape near nose or mouth
- Can be cut for emergency removal
- Color coded sizes
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

<i>Cat No.</i>	<i>Size</i>	<i>Color</i>	<i>Qty/Unit</i>
N709	Mini	Purple	5/box
N710	Ultra	Yellow	5/box
N711	Micro	White	5/box
N712	Small	Green	5/box
N713	Large	Blue	5/box
N714	XL	Rose	5/box
N715H	Macro	Clear	5/box
N716H	Jumbo	Gray	5/box
With Medical Grade Acrylic Adhesive			
N715F	Macro	Peach	5/box
N716F	Jumbo	Carbon	5/box
Assorted			
N718 Transport (1 each N709, N710, N711, N712, N713, N714)			
N719N Assorted (2 each N709, N710, N711, N712, N713, N714)			



U.S. Patent
#D434,496; #6,050,263

NeoBar®

ET Tube Holder

INDICATIONS FOR USE

The NeoBar is intended to secure an endotracheal tube. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Preparation

Step 1

After patient is intubated, select proper size NeoBar by using NeoBar measuring tape strip provided.

Place mid line of strip at septum and wrap to ear. Color that falls over opening of ear canal corresponds to NeoBar size.

Note: If size borders between two colors, use larger size.

Step 2

To ensure proper fit, position NeoBar across center of mouth, between upper and lower lips. NeoBar should not contact lips. Tabs must be just in front of ear.

Step 3

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1

Before removing liners, warm tabs in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

Step 2

Peel and discard liners. Apply tabs in front of ear on bone and hold in place for 60 seconds to ensure proper adhesion.

Note: Ensure ET tube is below platform to reduce pressure on palate or gums.

Step 3

Wrap 1/2" white or cloth tape completely around NeoBar platform first, then continue taping around both platform and tube.

Removal

Step 1

Replace NeoBar every 5 days or per hospital protocol, whichever is sooner.

Step 2

Unwrap tape to separate ET tube from NeoBar platform.

Step 3

Saturate tabs with water or saline.

Step 4

Slowly peel tabs away from skin as you swab with water or saline.

Emergency Removal

Carefully cut thin portion of NeoBar with blunt scissors at junction of bar and tab.

Remove NeoBar and attached ET tube.

Remove tabs later. (see Removal)

GENERAL INFORMATION

Product Materials

NeoBar: Polypropylene

Hydrocolloid tabs: NeoBond® Hydrocolloid, non-woven fabric

Acrylic tabs: Friendly foam, non-woven fabric, medical grade acrylic adhesive

Made with natural rubber latex: No

Made with plasticizer DEHP: No

Consult MSDS for complete material information.

Size

N709 Mini (Purple)

N710 Ultra (Yellow)

N711 Micro (White)

N712 Small (Green)

N713 Large (Blue)

N714 XL (Rose)

N715F Macro (Peach)

N715H Macro (Clear)

N716F Jumbo (Carbon)

N716H Jumbo (Gray)

Packaging

Individually packaged: Yes

Package type: Box

Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information or to download the Material Safety Data Sheet (MSDS).

Polyethylene bags are biodegradable. Boxes are printed with environmentally friendly soy ink on recyclable material.

CONTRAINDICATIONS, CAUTIONS & WARNINGS

The NeoBar is contraindicated for patients with infected or excoriated skin areas that would prevent the proper placement of the tabs.

It is not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoBar should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals.

Discontinue immediately if skin irritation occurs.

NeoBar should not contact lips.

Tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

For immediate removal, refer to DIRECTIONS FOR USE:

Emergency Removal.

Single patient use only.



Made in USA

For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products, Inc. • 28430 Witherspoon Parkway • Valencia, CA 91355