

# NeoBond<sup>®</sup>

## Hydrocolloid Adhesive Strips



### FEATURES AND BENEFITS

- Skin friendly alternative to acrylic tape
- Developed for long term attachment
- Ideal for use with limb boards
- Not made with natural rubber latex or plasticizer DEHP

Cat No.	Size	Qty/Unit
N741	1/2" x 6" Strips	100/box*
*50 sets of 2		
N742	1" x 15' Roll	1/box

# NeoBond®

## Hydrocolloid Adhesive Strips

### INDICATIONS FOR USE

---

NeoBond Hydrocolloid Adhesive Strips are intended for general use as a medical adhesive.

### DIRECTIONS FOR USE

---

#### Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

#### Application

##### Step 1

Before removing liner, warm NeoBond in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

##### Step 2

Peel and discard liner. Wrap NeoBond Strip around skin and product and hold in place for 60 seconds to ensure proper adhesion.

#### Removal

##### Step 1

Replace NeoBond Strip every 5 days or per hospital protocol, whichever is sooner.

##### Step 2

Saturate NeoBond Strip with water or saline.

##### Step 3

Slowly peel NeoBond Strip away from skin as you swab with water or saline.

### GENERAL INFORMATION

---

#### Product Materials

NeoBond Hydrocolloid, polyester silicone coating, polystyrene backing

Made with natural rubber latex: No  
Made with plasticizer DEHP: No  
Consult MSDS for complete material information.

#### Size

N741 pre-cut strip: 1/2 in. x 6 in.  
N742 roll: 1 in. x 15 ft.

#### Packaging

Pre-cut Strip: Box  
Strips of two on plastic liner.  
Roll: Dispenser box  
Continuous roll with coated paper liner.

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information or to download the Material Safety Data Sheet (MSDS).*

*Boxes are printed with environmentally friendly soy ink on recyclable material.*

### CAUTIONS & WARNINGS

---

Discontinue immediately if skin irritation occurs. NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.  
Single patient use only.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit [neotechproducts.com](http://neotechproducts.com)

Neotech Products, Inc. • 28430 Witherspoon Pkwy • Valencia, CA 91355, USA