

*New sizes*  
**NeoSeal**<sup>®</sup>  
Nasal Prong Seal



**FEATURES AND BENEFITS**

- Designed to improve the seal for more effective respiratory support
- Protects the skin of the septum and nares
- Compatible with most nasal prongs
- Not made with natural rubber latex or plasticizer DEHP

Cat No.	Size	Qty/Unit
N420	Micro	150/box
N421	Small	150/box
N422	Large	150/box



Made in USA

Patent Pending

# NeoSeal<sup>®</sup>

## Nasal Prong Seal

### INDICATIONS FOR USE

---

The NeoSeal is intended to create a seal around the nares and protect the skin of the septum and nares. It is intended for use on pediatric (neonates, infants and children) patients.

### DIRECTIONS FOR USE

---

#### Step 1

Peel NeoSeal from liner.

#### Step 2

Slip over nasal prongs with adhesive facing cannula.

Note: Soft, smooth side without adhesive should contact patient's skin.

#### Step 3

Place cannula per hospital protocol.

#### Step 4

Replace NeoSeal daily or per hospital protocol, whichever is sooner.

Note: Replace immediately if soiled or wet.

---

### GENERAL INFORMATION

---

#### Product Materials

PVC foam, PSA adhesive (does not contact skin)  
Made with natural rubber latex: No  
Made with plasticizer DEHP: No  
Consult MSDS for complete material information.

#### Size

N420 Micro: 20 mm x 10 mm  
Prong opening diameter: 4.5 mm  
Septal space: 4 mm

N421 Small: 25 mm x 12.5 mm  
Prong opening diameter: 5.5 mm  
Septal space: 5 mm

N422 Large: 30.5 mm x 12.5 mm  
Prong opening diameter: 6.5 mm  
Septal space: 6 mm

#### Packaging

Dispenser box: Yes  
Perforated liner

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information or to download the Material Safety Data Sheet (MSDS).*

*Boxes are printed with environmentally friendly soy ink on recyclable material.*

### CAUTIONS & WARNINGS

---

Federal law restricts this device to sale by or on the order of a physician.

Replace immediately if soiled or wet.

Discontinue immediately if skin irritation occurs.

Single patient use only.



**For more information or to order your FREE sample call  
1-800-966-0500 or visit [neotechproducts.com](http://neotechproducts.com)**

Neotech Products, Inc. • 28430 Witherspoon Pkwy • Valencia, CA 91355, USA