


NeoSmile™

Temperature Probe Cover

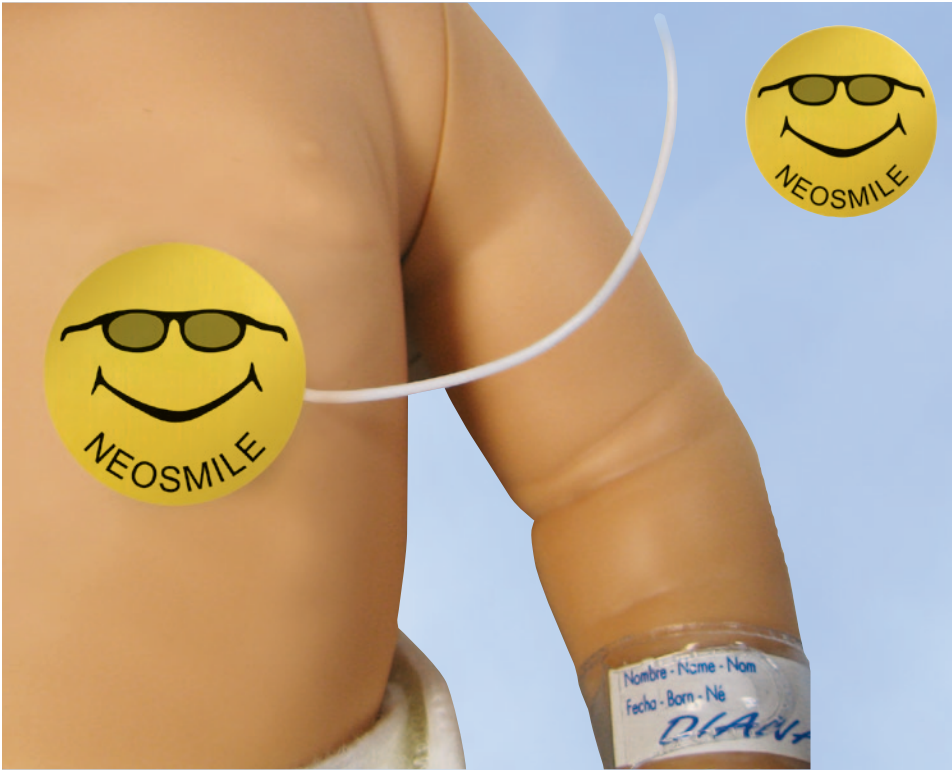

FEATURES AND BENEFITS

- Skin friendly NeoBond® Hydrocolloid base
- Designed for long lasting adhesion
- Reflective foil helps protect against ambient and radiant heat
- Cute smiley face design
- Not made with natural rubber latex or plasticizer DEHP

Cat No.	Size	Qty/Unit
N731	1" Diameter	150/box

NeoSmile™

Temperature Probe Cover

INDICATIONS FOR USE

The NeoSmile is intended to cover and secure a temperature probe.

DIRECTIONS FOR USE

Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1

Before removing liner, warm NeoSmile in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

Step 2

Peel and discard liner. Place NeoSmile over probe and skin and hold in place for 60 seconds to ensure proper adhesion.

Removal

Step 1

Replace NeoSmile every 5 days or per hospital protocol, whichever is sooner.

Step 2

Saturate NeoSmile with water or saline.

Step 3

Slowly peel NeoSmile away from skin as you swab with water or saline.

GENERAL INFORMATION

Product Materials

NeoBond Hydrocolloid,
FLEXmark® MM 200 gold foil
Made with natural rubber latex: No
Made with plasticizer DEHP: No
Consult MSDS for complete material information

Size

Diameter: 25 mm

Packaging

Standard box, no dispense feature

Perforated liner

Visit neotechproducts.com for the most up-to-date information or to download the Material Safety Data Sheet (MSDS).

Polyethylene bags are biodegradable. Boxes are printed with environmentally friendly soy ink on recyclable material.

CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Do not reuse or reapply.

Single patient use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products, Inc. • 28430 Witherspoon Pkwy • Valencia, CA 91355, USA